




AUTOFOLD

AutoFold Powerchair

QUICK START GUIDE


1. Unpack powerchair and stand on its legs so it is standing in an upright folded position (A). Cut the nylon tie from the battery (B). Remove the rubber block from the battery (C) then firmly push the battery backwards until it locks in position with an audible click.

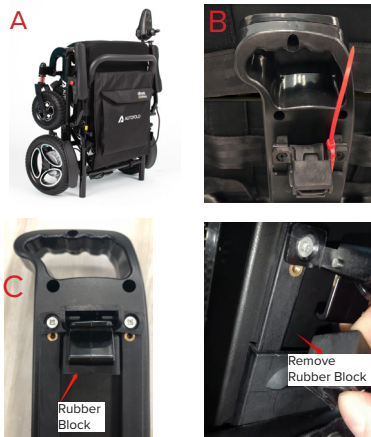

2. Install joystick by choosing preferred side, then turn the securement knob to loosen and then insert the joystick tubing. When the required position is reached, tighten the knob to secure. Connect the 2 terminals ensuring ends are locked in place.



3. Add the cable clips to the frame.

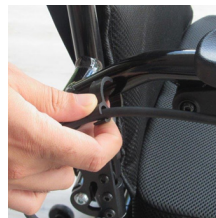

4. Lay the powerchair down, so that the four wheels are touching the floor.



5. Unfold the powerchair. At the rear of the chair, set the right hand switch to 'I'. On the left hand switch, continually press the side marked 'II', until the powerchair is fully unfolded. Return the right hand switch to 'O'.



6. Install battery in remote fob (CR2032). Open the battery compartment. Insert the CR2032 battery and then press together the plastic housing to lock. (Note: for some markets, the battery may already be installed).



7. Put freewheel levers into 'Electric' mode by pushing the levers forward. Repeat for both motors.



8. Drop down armrests and pull-down footrest.



9. Install the seat and back cushions which attach with Velcro (if necessary)



10. Charge battery. The battery can be charged via the powerchair by inserting the charger into the socket on the underside of the joystick. The battery can also be charged directly when removed from the powerchair. (See section 8.2 of the Instructions for use for more details).



Charging Socket

11. Switch the powerchair on. Ensure the charger is removed before operation.

12. Using the remote fob after initial set up.

To **unfold** the powerchair:

- Press
- Press
- Press and hold until motion has finished.

To **fold** the powerchair:

- Press
- Press
- Press and hold until motion has finished.



(See section 8.3 of the Instructions for use for more details).

To unfold powerchair



To fold powerchair



Follow these instructions to safely set up your new powerchair





Drive DeVilbiss Healthcare Ltd.
Sidhil Business Park, Holmfield, Halifax, West Yorkshire, HX2 9TN, GB

Tel: 0845 0600 333
Email: info@drivedevilbiss.co.uk
www.drivedevilbiss.co.uk

Drive DeVilbiss Sidhil Ltd.
4 Trench Road, Mallusk, Newtownabbey, BT36 4TY, UK



Sungo Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, NL



Sungo Certification Company Ltd
70 Gracechurch Street, London, EC3V 0HR, GB



Zhejiang Innuovo Rehabilitation Devices Co., Ltd.
196 Industry Road, Hengdian Movie Zone, Dongyang, Zhejiang, CN

Drive DeVilbiss Healthcare Ltd. provide this quick start guide.

NB. Images contained in this document are for illustration and parts identification purposes only, and may not reflect final product outlook appearance.



Issue: 2024-01 Rev5