

QUICK START GUIDE **CMS003 & CMS004 SCOOTERS**

1. Unpack all the parts.



2. Raise the tiller on the front section. Turn the tiller adjustment knob (1) anticlockwise to loosen. Lift up the tiller (2) to the required position and then turn the tiller adjustment knob clockwise to secure.



3. Connect the front and rear sections together. Hold the back of the rear section with one hand and lift the lever (3), on the front section, with the other. Hook the sections together and then release the lever, ensuring that they are securely locked together.



4. Fit the seat post. Remove the bolt from the seat post and then insert the post into the receiver in the scooter base. Insert the bolt and attach the nut. Secure the seat post using the hex key and spanner provided.



5. Fit the battery. Lift the battery into the recess in the scooter base and press down gently to ensure that it is securely connected.

Warning! The battery pack is heavy



6. Fit the seat. Insert the shaft, under the seat, into the seat post on the scooter base. Lift up the lever (4) and rotate the seat gently, until it locks into position.



7. Fit the basket. Align the brackets on the basket with the brackets on the tiller and then press down to secure.



8. Fit the armrests. Slacken the armrest adjustment knobs (circled) and then insert the armrests into the seat frame. Set the armrests at the required positions and then turn the armrest adjustment knobs clockwise to secure.



9. Engage drive mode. Ensure that the brakes are on and that the scooter is in 'Drive' mode, by pushing the freewheel lever backwards.



10. Charge the battery. Connect the round plug from the battery charger into the charging socket on the battery pack. Connect the charger to the mains and fully charge the battery, before first use of the scooter

Caution! Ensure that you fully read the instruction manual before operating the scooter.